
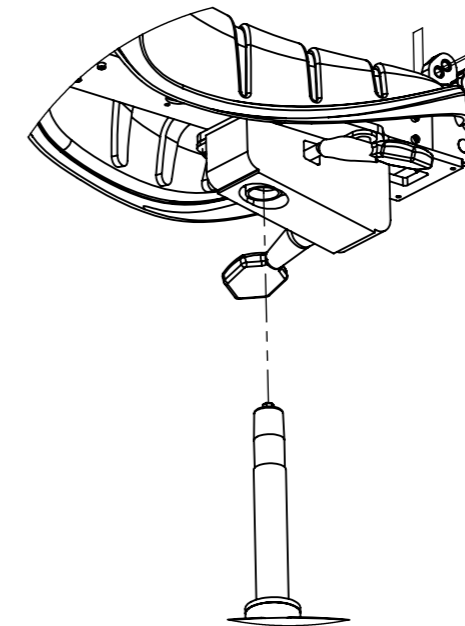



Assembly order:

1. Insert 4 x Interface bracket spacer (Item 4) into Interface bracket in Seat assy (Item 1) as shown in ASSY DETAIL 1-2.
2. Loosen the two grip knobs in the Seat assy (Item 1) (see ASSY DETAIL 1-1).
3. Add a drop of blue Loctite to 4 x threaded holes (see ASSY DETAIL 1-2)
4. Place Backrest (Item 2) between the two Interface brackets in the Seat assy. Fixate in center of brackets and finger tighten the two Grip knobs.
5. Loosen the Cam lever to release the Locking plate (see ASSY DETAIL 2).
6. Insert 4 screws M6x20 (Item 5) and torque to 10 - 12 Nm (see ASSY DETAIL 1-2).
7. Place the assembled Seat/ Backrest unit on top of the Undercarriage (Item 3) and press down to fixate.

REV.	DESCRIPTION	DATE
01	Notes added	23.02.2017
02	Drawing reorganized, detail views and Assembly order added, Item 4 revised	13.03.2017
03	AO edited, Detail view added	03.04.2017
04	Fastener (Item 5) updated	06.11.2017

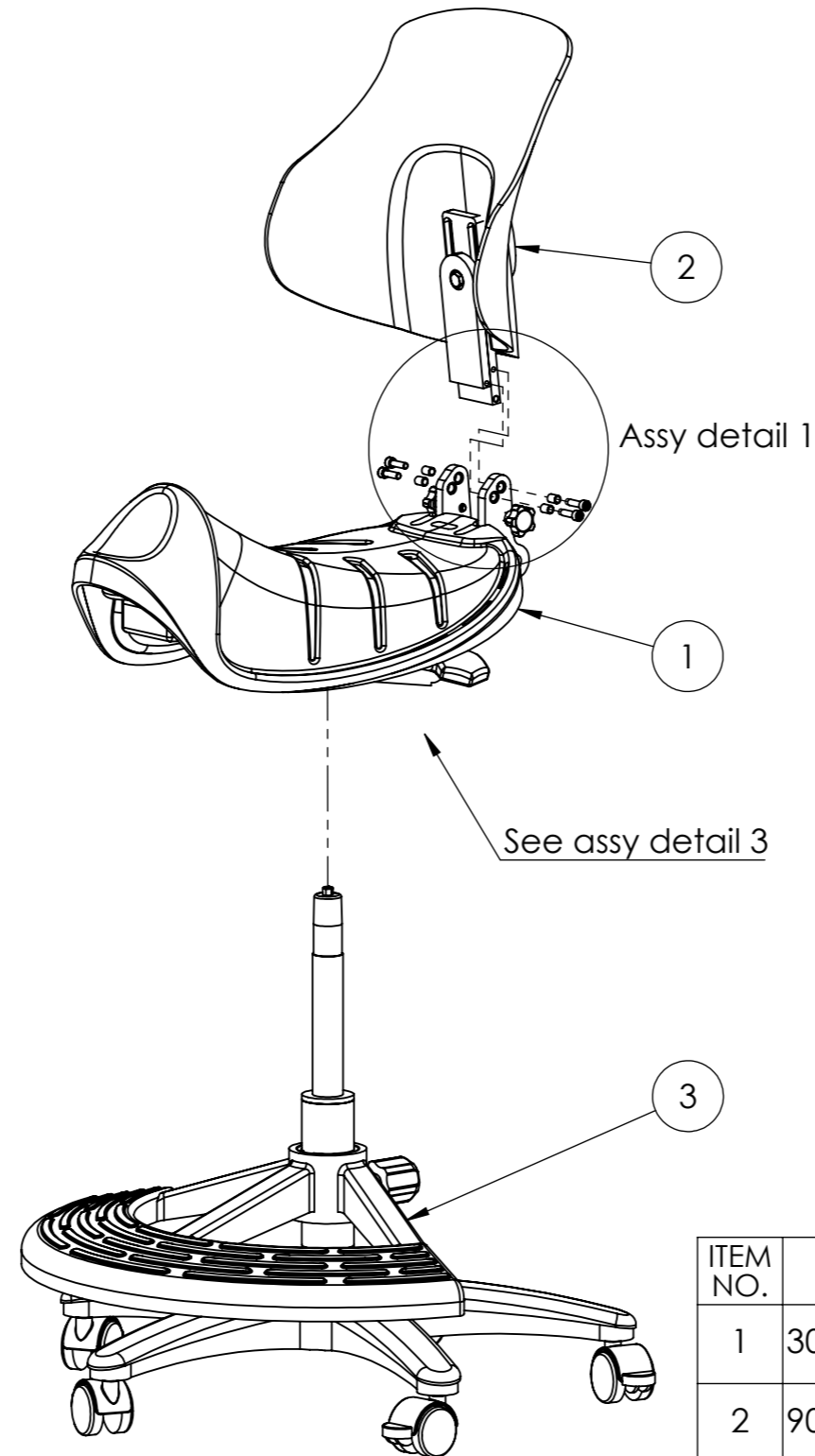


ASSY DETAIL 3
Seat to Undercarriage

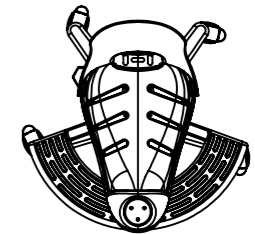


Align seat with undercarriage and Foot support as shown here


03

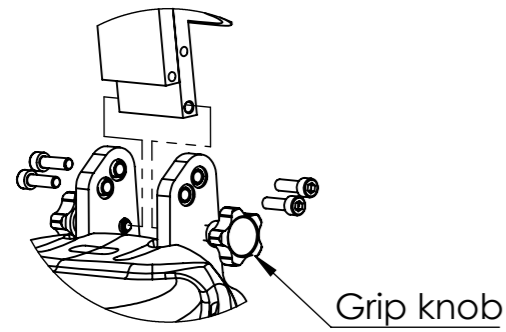

ASSY DETAIL 1-1
Positioning of Backrest

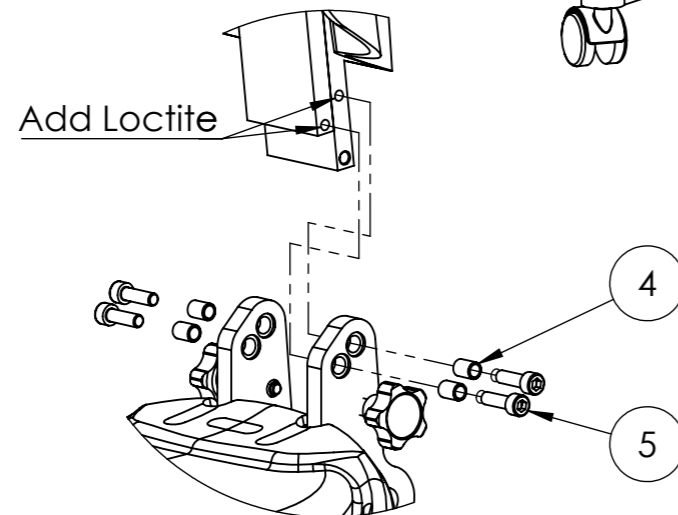

ASSY DETAIL 1-2
Backrest to seat

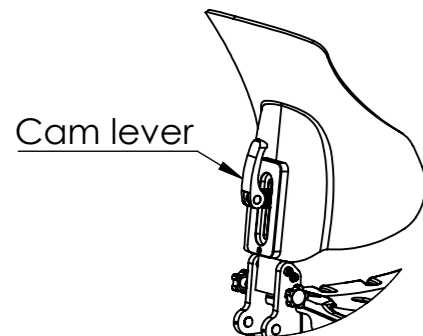

ASSY DETAIL 2

ITEM NO.	PART NUMBER	Title	QTY.
1	300-4014	Krabat Jockey Lite, Seat, complete, size 2	1
2	900-1135	Backrest, high, complete, size 2	1
3	300-4003	Jockey Lite undercarriage, size 1 and -2	1
4	900-1140	Interface bracket -spacer	4
5	DIN 912 M6 x 20 --- 20N - St - 10.9 - EZ		4

UNLESS OTHERWISE SPECIFIED:				<p>Krabat Jockey</p> <p>TITLE: Jockey Lite size 2, complete w/ footrest</p>	
ALL DIMENSIONS ARE IN MILLIMETERS	TOLERANCES, LINEAR/ ANGULAR: NS-ISO 2768-1-m-K	DEBUR AND BREAK SHARP EDGES	PAPER SIZE: A3		
		MATERIAL:	TREATMENT:	DWG NO. 300-4002	REVISION 04
NAME	SIGNATURE	DATE	FINISH:	SHEET 1 OF 1 SCALE:1:20 DO NOT SCALE DRAWING	
DRAWN A.S.		09.01.17	WEIGHT:		
CHK'D					
APP'VD					