


## Splitt tegninger

### Luna




## 1. Hjulkruss 2020

Hjulkruss 2020 blir brukt på Aktiv, Luna, Tilto stolene.

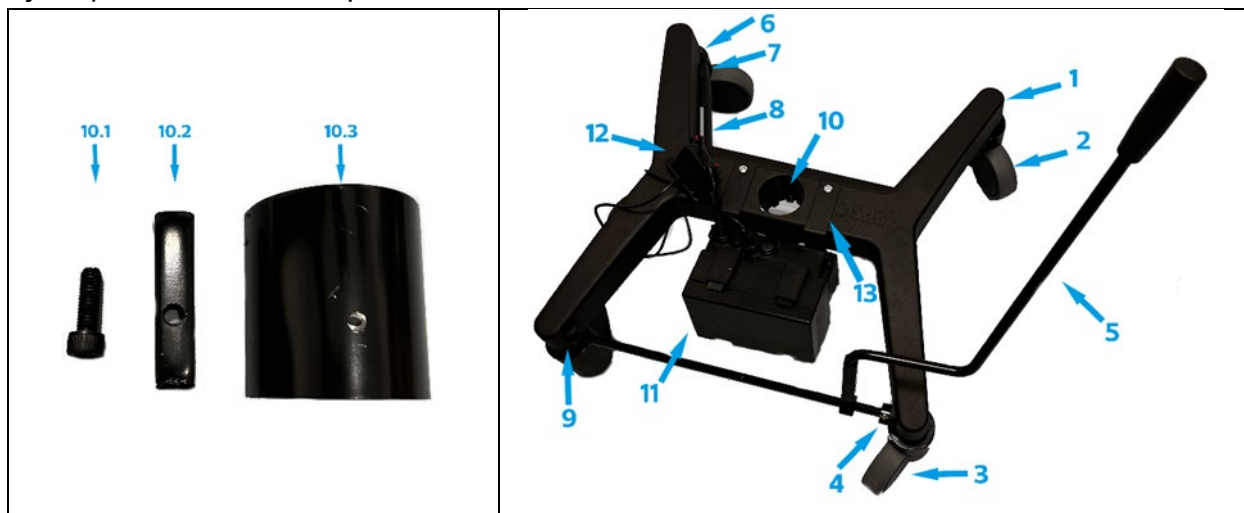

Pos.	Benevning	Antall	Hepro art. nr.	HMS art. nr.
-	Hjulkruss komplett Luna 2020	1	32602	-
1	Hjulkruss Luna 2020	1	32603	32001
2	Trinse Integral u/brems	2	31742	-
3	Trinse Integral m/brems	2	31743	-
4	Stopper 6-k stang brems 2019	1	32039	-
5	Bremespake, komplett	1	32605	-
6	Skrue innv. 6-kt., UBL M6x10	2	70145	-
7	Feste til 1/4-fotring 2004 m/foring og skrue	2	30878	-
8	1/4-fotring til 32602	1	32002	32604
9	Skrue innv. 6-kt., UBL M8x16	2	70154	-
10.1	Festeklemme skrue	1	32005	-
10.2	Festeklemme lock	1	32005	-
10.3	Festeklemme til sentralenhet	1	32005	-
<b>Kun E2 modeller</b>				
11	El.skap m/batteri Luna	1	31403	-
12	Bryterboks kompl. m/kabel L120	1	31954	262441
13	Festeplate til batteri Luna E	1	32553	-

## 2. Løfteenhet

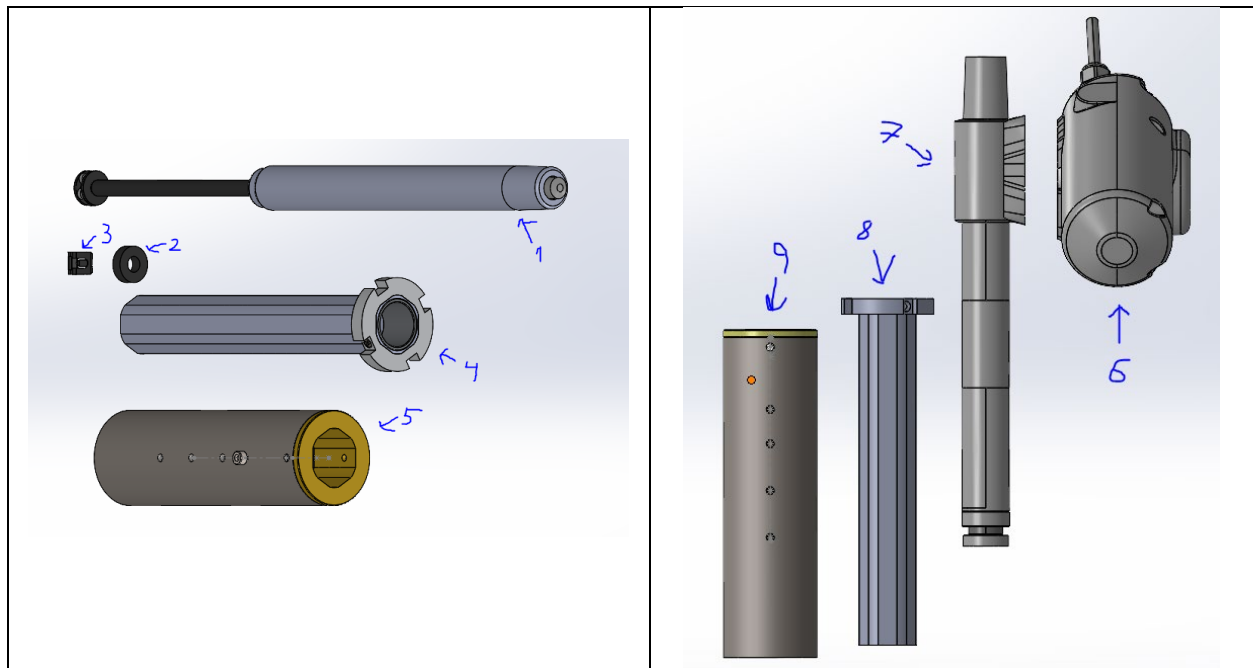

Pos.	Benevning	Antall	Hepro art. nr.	HMS art. nr.
<b>G2 modeller</b>				
-	Sentralenhet G150 2011	1	31850	-
1	Gassfjær 160, 400N - ROF	1	32270	32607
2	Gummidemper til gassfjær	1	70232	-
3	Låsebrikke til gassfjær	1	70237	-
4	Sentralenhet G150 2011 - Styringstube	1	31915	-
5	Sentralenhet G150 2011 - Tube	1	31915	-
<b>E2 modeller</b>				
-	Aktuator	1	61671	-
6	Motordel LA30.L	1	61543	-
7	Spindel LA30. 3M-150 m/kon	1	61537	-
8	Sentralenhet E150 2011 - Styringstube	1	31855	-
9	Sentralenhet E150 2011 - Tube	1	31855	-

### 3. Setebeslag




Pos.	Benevning	Antall	Hepro art. nr.	HMS art. nr.
-	Setekryss Luna G2/E2	1	32610	-

### 4. Armlene





Pos.	Benevning	Antall	Hepro art. nr.	HMS art. nr.
-	Armlene Royal Høyre	1	31689	192230
-	Armlene Royal Venstre	1	31680	192231

## 5. Sete

1.		2.		
				
Pos.	Benevning	Antall	Hepro art. nr.	HMS art. nr.
1	Setepute Luna Liten	1	32646	292211
2	Setepute Luna Stor	1	32647	292212

## 6. Rygg

1.		2.		
				
Pos.	Benevning	Antall	Hepro art. nr.	HMS art. nr.
1	Ryggpute Luna Liten	1	32638	292210
2	Ryggpute Luna Stor	1	32636	292205