# Hepro Ståstoler

BRUKSANVISNING/USER MANUAL

NORSK

ENGLISH

CE

HEPRO

# Advarsel

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# Warning

This product is intended for indoor use in private homes or institutions.

- Read instructions for use before use. Keep this manual in a safe place.
- Do not use the chair outdoor.
- Always consider the risk of clamping between the moving functions of the product.
- Do not place on surfaces that are not flat.
- Do not roll the chair over cables and / or cords.
- Do not stand on the chair.
- Do not use the chair when it has defects.
- Do not use the chair when the upholstery is damaged.
- Do not modify or otherwise alter the product in ways not described in this manual.
- Do not use this product with accessories other than those described in this manual.
- Do not immerse the product in water or other liquids.
- Do not wash the product with dishwasher, chlorine, or acetone.
- Be careful when opening the packaging so that neither you nor the product is damaged.
- The chair is an aid and should not be used as a "toy" for children.
- The chair shall only be used by one person at a time.
- Exceeding the approved user weight (120 kg) can damage both the user and the product.
- Even if the fabric is flame retardant, it must not be exposed to open flame, cigarettes, or the like.

Remember to read the instructions carefully before using the chair!

This manual is written for you to get acquainted with your new Hepro product and applies for the following adult models:

Hepro art.	Modell
32950	Hepro G2 Ergo Liten
32951	Hepro G2 Ergo Stor
32952	Hepro G2 Sadel Liten
32953	Hepro G2 Sadel Stor

The operating instructions contain useful tips on the use and setting of the work chair, warnings related to the use of the product, as well as information on cleaning and maintenance routines for to ensure that you get the most out of your chair.

# Table of contents

Symbols .....	12
Intended use.....	13
Contents of the package.....	13
Transport.....	13
Recycling.....	13
Dimensions.....	13
Adjustments.....	14
Cleaning. ....	16
Maintenance.....	16
Service and warranty.....	17

# Symbols

Producer

Warning

Production Date

Read the User Manual before use

Serial Number

CE-marking

The product can be sorted at source

The product is a medical device

\* Label is located on the seat mechanism.

## Intended use

The intended use of the chair is to support the user in a sitting posture while performing indoor activities like work or day-to-day chores.

The chair has an maximum user weight of 120 kg.

The product is designed, manufactured and tested according to relevant parts of:

- EN 1335-2:2018
- EN 1021-1:2014
- EN 1021-2:2014

## Contents of the package

Hepro work chair consist of:

- 1x Backrest
- 1x Seat
- 1x Seat mechanism
- 1x Gaslift
- 1x Wheel base
- 5x Castors

The chair comes assembled.

## Transport

During transport, the product must be secured so that it cannot pose a danger to passengers or other objects.

## Recycling

All parts should be recycled in accordance with local regulations, please contact your local recycling station.

Gaslift

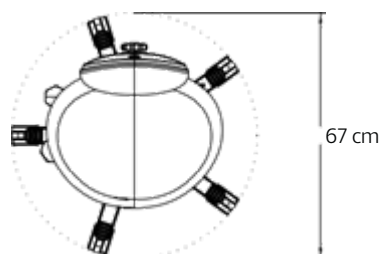
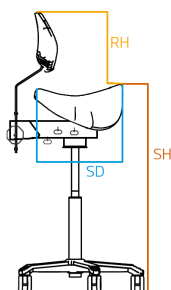
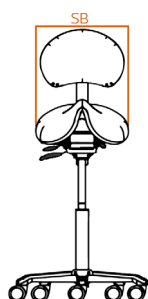
Wheel base  
(including  
castors)

Backrest, Seat and Seat  
Mechanism

## Dimensions

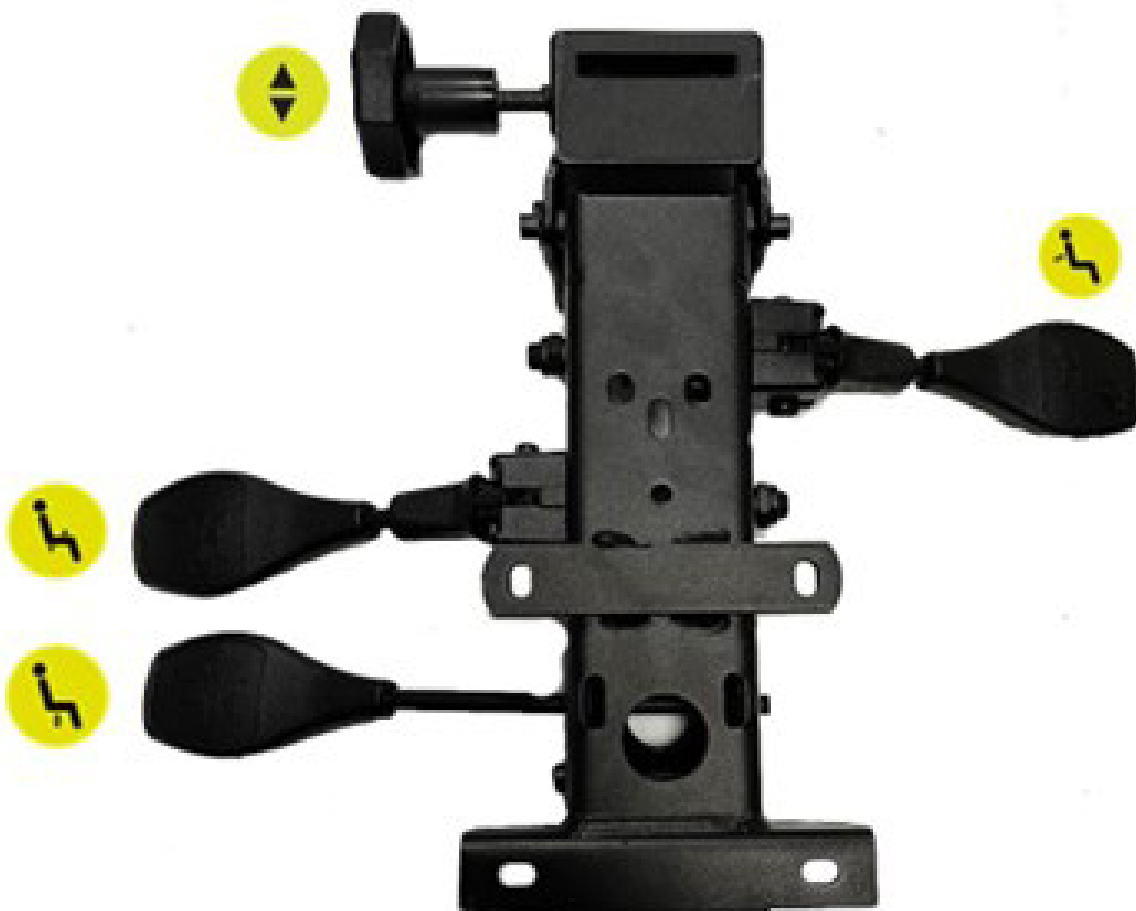
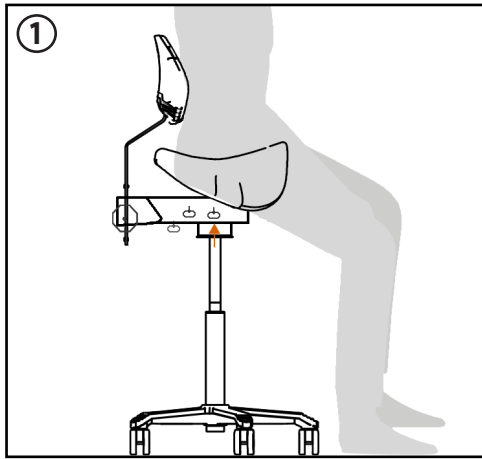
Hepro art.	Modell	Seat width (SB)	Seat depth (SD)	Seat height (SH)	Backrest height (RH)
32950	Hepro G2 Ergo Liten	35 cm	35 cm	59-78 cm	24-40 cm
32951	Hepro G2 Ergo Stor	40 cm	40 cm	59-78 cm	24-40 cm
32952	Hepro G2 Sadel Liten	34 cm	33 cm	60-79 cm	23-38 cm
32953	Hepro G2 Sadel Stor	44 cm	33 cm	60-79 cm	23-38 cm

# Adjustments


Take time to get acquainted with the chair's functions so that you can adapt it to your needs in the best possible way. Your height, weight, physical strength and function can affect your needs for adjustments to the various functions. Consult an occupational therapist if you need help to achieve a good and ergonomic sitting position, tailored for you.

-  Backrest: Backrest height is adjusted to achieve the best possible lower back support. Turn the knob at the back of the seat mechanism to adjust the height.
-  Backrest angle: Back angle adjusted for best support and comfort.
-  Tilt: The tilt function provides 3° forward tilt that makes it easier to move with the chair. Also, the tilt function makes it possible to tilt the chair 8° back from the starting position.
-  Height adjustment: Adjust the chair to your own height for the best seating ergonomics.


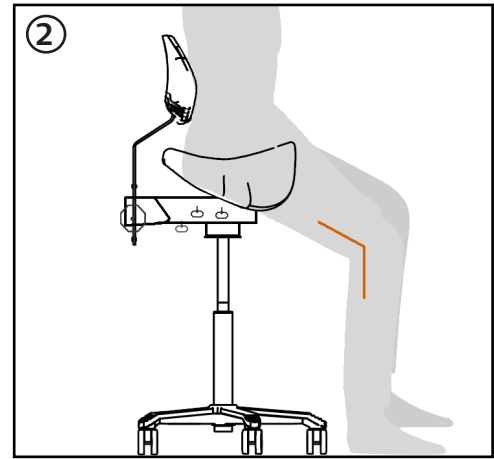
Adjust the seat height to a comfortable height. Most forward lever on right side.

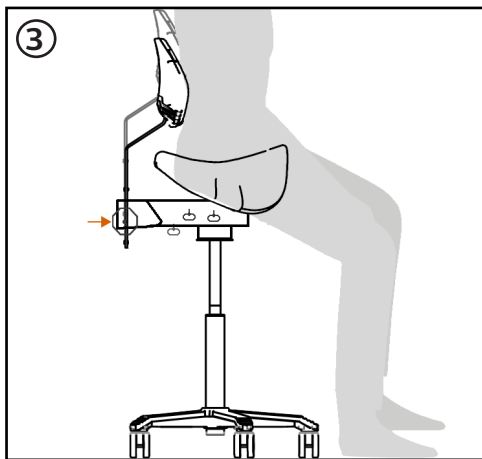
Optimal height gives a approximately. 110 degree angle with the legs.

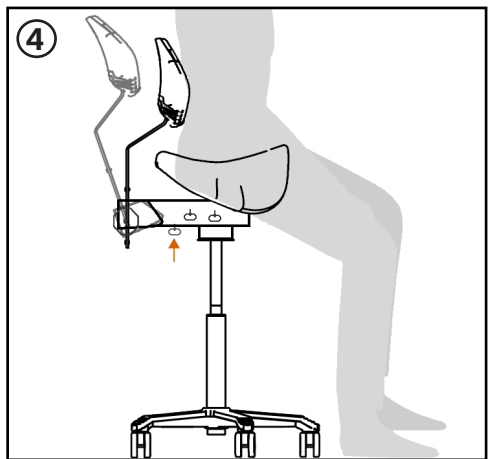
Adjust the depth of the pelvic support so that the front of the support makes contact with the lower back.

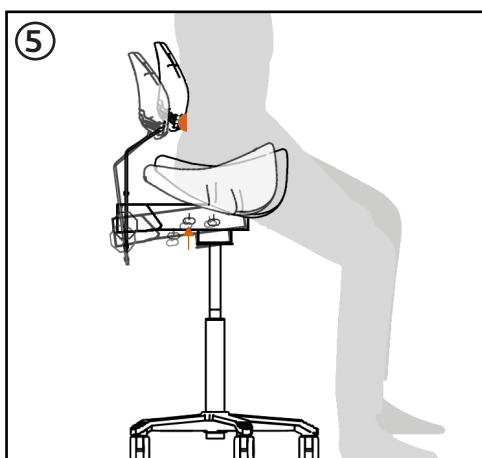Adjust the tilt on the pelvic support so that it is comfortable to your use. Lever is on the left side.

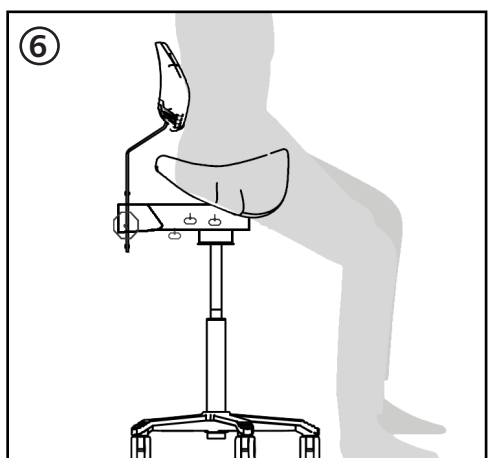Adjust the seat tilt so that it fits your use. Furthes back lever on right side.

Note: Make sure the pelvic support makes con- tact and does not give pressure. If the pelvic support does give pressure, readjust the depth.

## Cleaning

### Cleaning synthetic parts and base

The synthetic parts, base and castors can be cleaned with a damp cloth and if necessary with a mild detergent without scouring powder or an approved disinfectant. After cleaning, please ensure your product is wiped dry. Never wipe the piston rod of the gaslift (height adjustment) with water or any other cleaning detergent.

### PUxx

Clean with a damp lukewarm/warm cloth, use a cleaning agent if necessary. PUxx is resistant to most cleaning and disinfecting agents based on 70% alcohol, chlorine or ammonia. Follow the instructions of the disinfectant manufacturer. Always wipe the cleaning and disinfecting products with a clean damp cloth. Do not use solvents or abrasives. After cleaning, dry the upholstery with a dry cloth.

## Wheel

Make sure that dust and hair do not accumulate between the wheels, as this may impede the quality of the roller.

Approved disinfectants:

- Alcohols
- Chlorine compounds

See the complete list of approved disinfectants at <http://www.legemiddelverket.no>.

## Maintenance

Periodically check the product for function and for damage. Periodic checking contributes to a longer service life and trouble-free use. Reduced functionality may indicate future problems. Is a component damaged or is its operation impaired? If so, please contact Hepro AS

Hepro G2 Ergo/Sadel is designed and tested for a normal lifespan of ten years.

Check the entire product's functionality at least once a year. Damaged parts should be replaced. Contact Hepro AS for repair and/or replacement of parts.

## Products with special loads

If the product is subjected to special loads, it will be an advantage for everyone that in the individual case it is considered and necessary inspection is established.

Hepro will provide advice and guidance if necessary.

# Service and warranty

## Service

When contacting Hepro AS regarding defects or accessories, the serial number must always be stated. Get in touch on [service@hepro.no](mailto:service@hepro.no).

The product should be suitable for its purpose, have excellent quality and performance. For valid warranty enquiries, Hepro will, in its sole discretion, replace, repair or refund parts that we agree are defective.

## Warranty

For use in accordance with the operating instructions, HEPRO provides a 1 year warranty against material and manufacturing faults. In the event of complaints or inquiries about the product, the serial number must always be given as a reference.

In some cases, a separate agreement applies.

Any damage or defects from improper use, improper operation or incorrect storage are not covered by warranty. The warranty should start from the time the product is delivered.

For service and warranty cases, the following applies: The customer must contact Hepro at [service@hepro.no](mailto:service@hepro.no).

# Notater

# Notes

# HEPRO

Aktiv og trygg  
i alle livets faser