# HEPRO

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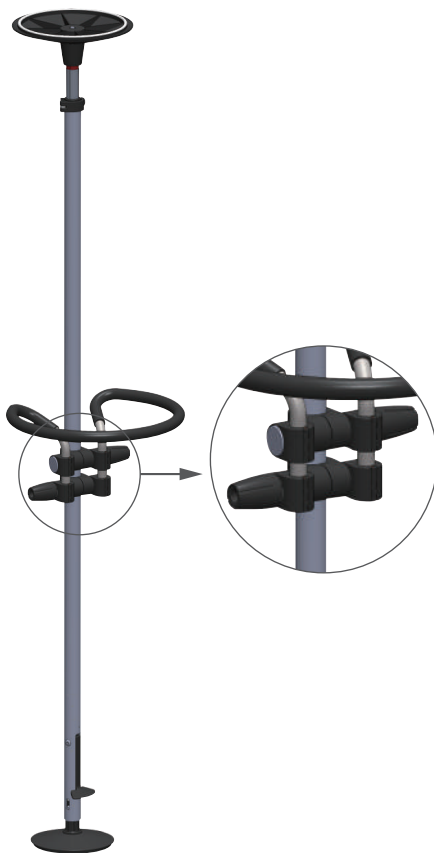
Artnr. 35280 / 35281

## A

Unwrap the recieved parts and make sure nothing is missing

It should consist of:

- **A1** 2 x Universal connectors 22/40/22 mm
- **A2** Circular grip D40 or D30

**A1**

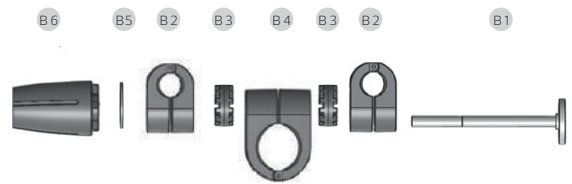
**A2**

## B


The universal connector is disassembled, into eight parts.

## C

Attach the universal connector **A1** to the Gripo pole:  
This is done by attaching the largest clamp **B4** to the pole.

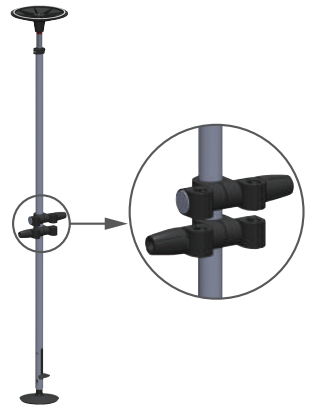1. Thread **B2** together with **B3** on to **B1**, and then thread this through **B4**.
2. On the opposite side of **B4** thread on **B3**, **B2**, **B5** and at the end put on **B6**.
3. Angle part **B2** so that the accessories fit.
4. Find the desired height and tighten so that the universal connectors holds, but loosely enough that the accessories can be threaded on as described in the next step.


## D

When both universal connectors (A1) is assembled on the Gripo and adjusted to the desired height, insert the ends of the circular grip inside B2 on the universal connector (A1) that is at the top.

When the circular grip is inserted in the upper universal connector (A1), push the universal connector (A1) at the bottom upwards until it covers the ends of the circular grip.

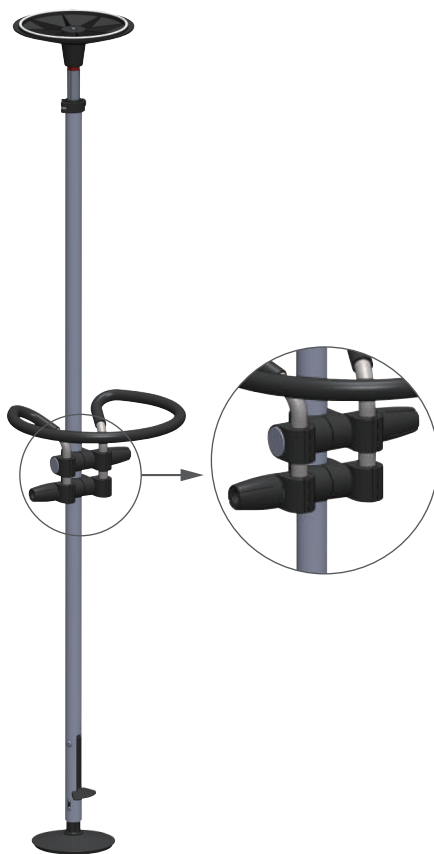When the ends are inserted in both universal connectors, use the hand-wheel (B6) to thighten them to secure that the accessorie is properly fastened and safe to use.

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