HEPRO

SENGEHEST

Bruksanvisning

| | |
|---------|---------|
| NORSK | 2 - 8 |
| ENGLISH | 9 - 16 |
| SVENSKA | 17 - 24 |

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HEPRO

BED RAIL USER MANUAL

NB! This accessory is designed to prevent the user from falling out of bed and to prevent the risk of entrapment between Gripo and the bed. Therefore, read the user manual carefully before assembly to ensure that the installation is carried out in accordance with the manufacturer's recommendations.

Assembly and use of BED RAIL

Art. Nr. 35326

Assess the distance between the bed rail and any movable furniture, as well as the stability of the mattress during installation.

A

Unpack the received parts and check that nothing is missing.

It should consist of:

- A1** – 1 short end piece
- A2** – 1 long end piece
- A3** – 2 middle angles / centre brackets
- A4** – 4 screws
- A5** – 1 Allen key
- A6** – 2 universal clamps 22/40 mm
- A7** – 1 cover (MUST ALWAYS BE USED)

A7

A6

A3

A4

A2

A5

1. Slide the **A3** middle bracket into the short end piece **A1** and the long end piece **A2** so that the assembly looks like in figure 1. Make sure that the **A3** middle bracket is slid on so that the holes are on the inside of the bed rail, as shown in figure 2. This is to prevent the screw heads from obstructing when the cover is pulled on.

2. Join the parts together using **A4**, four screws. First fasten the screws loosely. When all the screws are in place, tighten them so that there is no risk of the screws coming loose.

(figure 1)


(figure 2)

C

Unscrew the universal connector into six separate parts.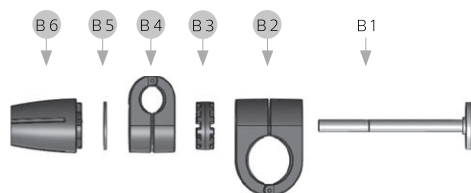

D

1. Position the first clamp **B2** around the Gripo pole at the desired height.
2. Place clamp **B4** onto one tube of the shortest side of the bed rail.
3. Choose the side for the locking knob **B6** so that it causes minimal obstruction for the user
4. Assemble the universal clamp in the reverse order of step **C**. Leave the clamp slightly loose to allow for final adjustment.

E

Tilt the bed rail away to gain access for installing the next clamp.


Repeat the procedure described in step **D**.

F

Adjust the distance between the bed rail and the bed by sliding the short end of the bed rail in the desired direction.

When the desired distance has been achieved, firmly tighten the universal connectors.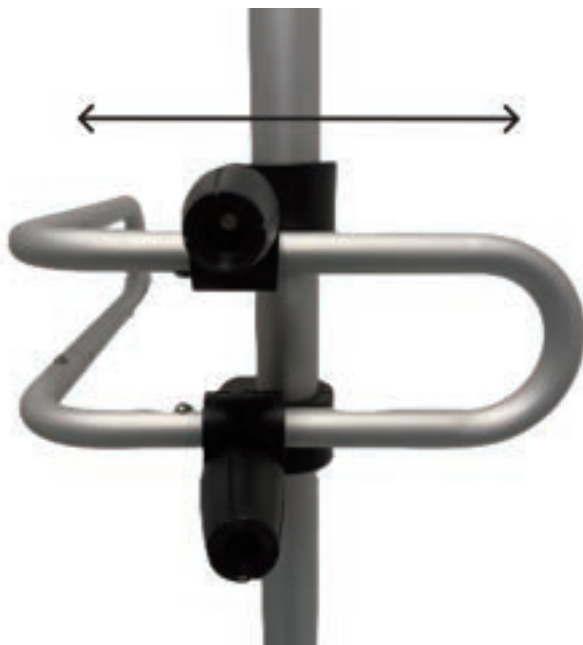


During installation of the bed rail, you must continuously assess which distance will present the lowest risk of entrapment or pinch injury. Always assess the risk of entrapment between the bed rail and the bed, and pay particular attention to adjustable or movable parts and objects that may change this distance. Make sure that the recommendations below are taken into account during installation.

Safety during installation – Warning of risk of entrapment and pinch injury

Use of the bed rail may involve a risk of entrapment and pinch injury between the bed rail and the bed/mattress. Distances that require particular attention are:

- the distance between the lowest tube of the bed rail and the bed frame/mattress
- the distance at the head end where the bed rail is mounted
- any excessive gap between the bed rail and the bed/mattress

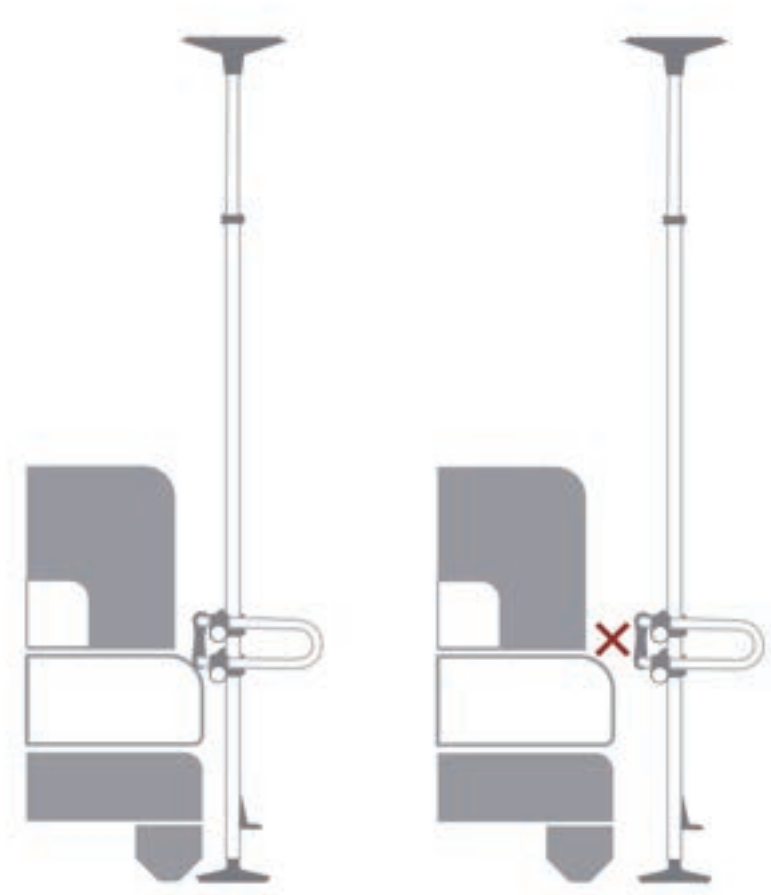Special considerations when used in combination with soft or thick mattresses:

Use of soft or thick mattresses (e.g. foam mattresses, pressure-relieving mattresses or mattress toppers) may change the distance between the bed rail and the mattress. It is therefore important that the following precautions are followed:

- Check that there are no openings or gaps that would allow the head, neck, arms or legs to become trapped between the bed rail and the mattress.
- Check the stability of the mattress. Ensure that the mattress does not sink so far down that a gap is created under the lowest tube, allowing body parts to become trapped.
- Whenever a new or softer mattress is used, all distances and gaps must be checked again

Special considerations when used in combination with a movable bed or height-adjustable bed:

- Check the distances between the bed rail and the bed in both the highest and lowest bed positions, and inform users/care staff that they must pay particular attention when the bed is moved.
- Always make sure that no body parts (hands, arms, legs, head) are between the bed rail and the bed frame/mattress when the bed is raised, lowered or moved.
- After any adjustment of the bed height or repositioning of the bed, check the distances and openings between the bed rail and the bed again.

G

When the bed rail has been mounted at the desired height, pull the cover over the long side of the bed rail and fasten it on the inside using the hook-and-loop closure to prevent it from sliding off. The cover must always be used to ensure correct function of the bed rail and to protect the user from entrapment and pinch injuries.

Washing instructions for the cover (98% polyester, 2% Spandex):

- Machine wash at 30 °C
- Air dry (do not tumble dry)
- Do not dry clean
- Maximum ironing temperature 110°C

NB! Always make sure that no body parts are between the bed rail and the mattress/bed frame before moving the bed or adjusting the bed height.

NB! Always check that no new gaps have formed between the bed rail and the bed after changing the mattress or adjusting the bed height.

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