HEPRO

GRIPO

Manual

NORSK	2 - 6
ENGLISH	7 - 11

Symbolforklaring / Explanation of symbols

Produsent /
Manufacturer

Advarsel /
Caution

Produksjonsdato /
Production date

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Instruction Manual

GRIPO

Built and produced in accordance with:

- Essential Requirements of EU Directive 93/42/EEC (CE mark)
- Relevant parts of EN 12182

INSTRUCTIONS BEFORE USE

Specially adapted information

For information adapted for users with visual impairments, reading or writing difficulties and cognitive impairments; please contact your sales representative or therapist.

Environment of use

This product is intended for indoor use in private homes or institutions. The Gripo pole must be at room temperature before instalment.

User weight

Maximum user weight is 150 kg.

Maintenance

It is recommended that the pole is disassembled twice a year for the gas spring to maintain its functionality.

Cleaning

All parts can be cleaned with domestic cleaning products, at a maximum of 60°C. Avoid strong solvents.

Dimensions

Weight	4,5 kg
Height	see table next page
Breadth base plate	290 mm

NB! The instruction manual is only valid at the time of the printing. The latest version will always be available at our website.

Transportation

During transportation, the product must be secured so that it cannot cause damage to passengers or other objects.

USER INFORMATION

Placement of serial number

The serial number is placed at the bottom of the product's outer aluminium tube.

Intended use

Intended use for persons in need of assistance or support for the performance of everyday movements.

Recycling

Pole	Aluminium
Black plastic parts	PA6
White rubber rings	Rubber
Other parts	Steel, copper and aluminium

All parts must be recycled in accordance with local regulations; please contact your local recycling plant.

Gripo


Art. No. 35100

Safety

This product is intended for indoor use in private homes or institutions.

- Do not use the product if the red safety ring at the top of the pole is visible.
- Always consider pinch hazard between the pole and other furniture/objects. Please keep in mind the distance between the pole and other furniture, and assess distance so that the user can not be pinched between the pole and the furniture.
- Do not assemble the product on loose objects that are not fixed to ceiling or floor, such as rugs, plates and the like.
- Do not let children play with the product.
- Do not place on surfaces that are not plane and parallel.
- For installation in wet zones, use anti slip underlays.
- Do not modify or in any other way alter the product in a manner not described in this Instruction Manual.
- Do not use this product with any additional equipment not described in this Instruction Manual.
- Do not submerge the product in water or other liquids.
- Do not wash the product with washing-up liquids, bleach or acetone.
- Take care when opening the package, so that neither you nor the product is harmed.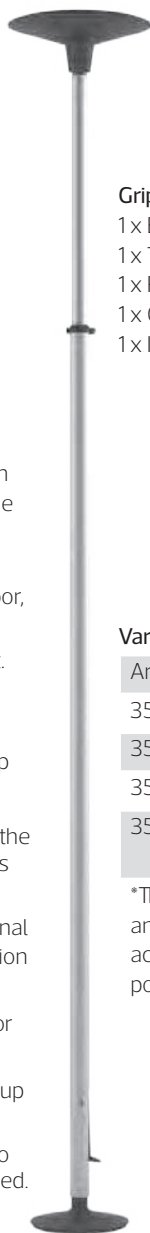

Gripo consists of:

- 1 x Base plate (the smallest)
- 1 x Top plate (the largest)
- 1 x Pole
- 1 x Gas spring
- 1 x Instruction Manual

Variations

Art. No.	Description	Ceiling height
35100	Gripo	210-300 cm
35105	Gripo, Long	260-320 cm
35106*	Gripo, Plus	225-300 cm
35202	Gripo, Special	Less than 210 cm

*This edition has 15 cm longer outer tube and is suitable for those who need the accessories attached far atop the pole, possibly use the pole for workouts

Assembling


GRIP0

Art. No. 35100

Art. No. 35105


Art. No. 35106

Art. No. 25202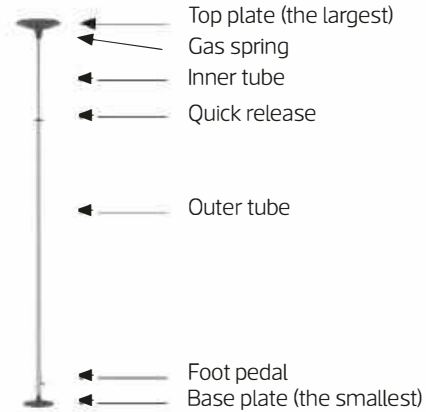

Installing Gripo against a ceiling

Before Gripo is installed, the type of ceiling in question must be determined and make sure that the ceiling is solid and can withstand a point load of 100kg. If the ceiling is made from gypsum, it is important that Gripo is assembled as shown in the illustration. Avoid placing Gripo between ceiling beams also if the ceiling is made from chipboard.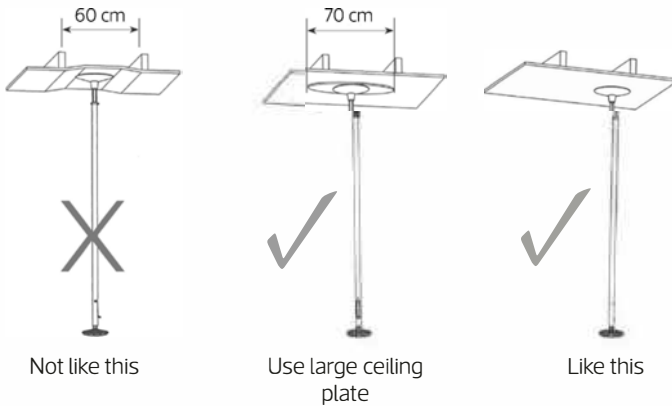


1

Place the pole on the small base plate.

Place the gas spring at the top of the inner tube and push the gas spring towards the red safety mark.

Place the large top plate on top of the gas spring.

 The force in effect against the ceiling is 100 kg.

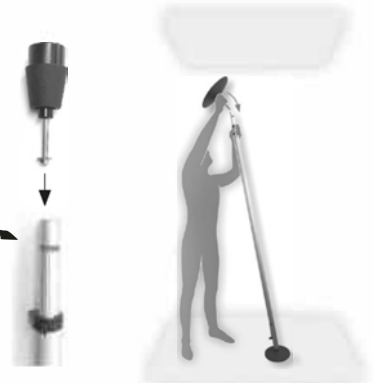2

Place the pole in a suitable place.

3

Pull out the foot pedal.

4

Open the quick release.

5

Slide the inner tube up until the top plate reaches the ceiling, while the bottom plate remains in contact with the floor.

Make sure the pole is vertical.

6

Rotate the inner tube clockwise so that the inner tube is locked in place with the outer tube.

7

Lock the quick release.

8

Press down the foot pedal. If there is any uncertainty regarding the strength of the ceiling, observe that the ceiling does not crack while slowly pressing down the foot pedal.

9

Make sure that the red mark is gone.

If the red mark shows, disassemble the pole (see next page) and start from point 2.

During use, make sure that the red mark is not visible. If it appears - disassemble the pole and reassemble it. Remember to rotate the inner tube as described in point 6.

Disassembling

Open the quick release (figure 4), and pull out the foot pedal (figure 3). If necessary, rotate the inner tube anticlockwise.